

CENTRI®

A Fillauer Company



Dream skin

Centri AB • Kung Hans Väg 2 • 192 68 Sollentuna • Sweden
Phone +46 (0)8 505 332 00 • Fax +46 (0)8 505 332 05 • www.centri.se

STANDARD

Features

- ▶ Natural appearance
- ▶ Durable
- ▶ Good tear resistance
- ▶ Low fabric friction
- ▶ Dirt and stain resistant
- ▶ Water resistant

Ordering Number / Item code for Standard model

Ordering Format	X	XXXX	X	XX	X
Dream Skin	4				
Foot Size		XX-XX			
Left Side			1		
Right Side			2		
Color (See Centri Color Guide 01-18)				XX	
MCV Expanded					4

Example: 4 2324 2 04 4




Standard



Color Guide
01 - 18



		Measurement chart			
Foot Size (A)	Side	Calf Size min (B)	Ankle Size min (C)	Length (D)	
21-22 cm	Left	30.5 cm	19.0 cm	500mm	
21-22 cm	Right	30.5 cm	19.0 cm	500mm	
23-24 cm	Left	32.0 cm	20.0 cm	550mm	
23-24 cm	Right	32.0 cm	20.0 cm	550mm	
25-26 cm	Left	34.0 cm	21.0 cm	600mm	
25-26 cm	Right	34.0 cm	21.0 cm	600mm	
27-28 cm	Left	35.5 cm	21.5 cm	550mm	
27-28 cm	Right	35.5 cm	21.5 cm	550mm	
29-30 cm	Left	37.0 cm	22.0 cm	600mm	
29-30 cm	Right	37.0 cm	22.0 cm	600mm	

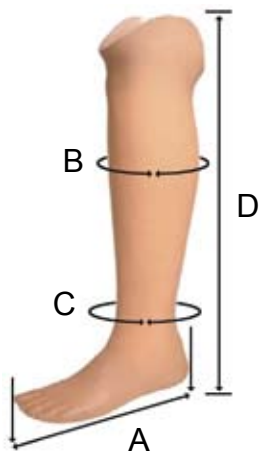



Split Toe



Color Guide
01 - 18



Features

- ▶ Natural appearance
- ▶ Durable
- ▶ Good tear resistance
- ▶ Low fabric friction
- ▶ Dirt and stain resistant
- ▶ Water resistant

SPLIT TOE


Ordering Number / Item code for Split Toe model

Ordering Format	X	XXXX	X	XX	X
Dream Skin	4				
Foot Size		XX-XX			
Left Side			3		
Right Side			4		
Color (See Centri Color Guide 01-18)				XX	
MCV Expanded					4

Example: 4 2324 4 04 4

└─ MCV Expanded
└─ Color
└─ Right Side
└─ Foot Size
└─ Dream Skin

		Measurement chart			
Foot Size (A)	Side	Calf Size min (B)	Ankle Size min (C)	Length (D)	
21-22 cm	Left	30.5 cm	19.0 cm	515mm	
21-22 cm	Right	30.5 cm	19.0 cm	515mm	
23-24 cm	Left	32.0 cm	20.0 cm	550mm	
23-24 cm	Right	32.0 cm	20.0 cm	550mm	
25-26 cm	Left	34.0 cm	21.0 cm	600mm	
25-26 cm	Right	34.0 cm	21.0 cm	600mm	
27-28 cm	Left	35.5 cm	21.5 cm	620mm	
27-28 cm	Right	35.5 cm	21.5 cm	620mm	
29-30 cm	Left	37.0 cm	22.0 cm	680mm	
29-30 cm	Right	37.0 cm	22.0 cm	680mm	

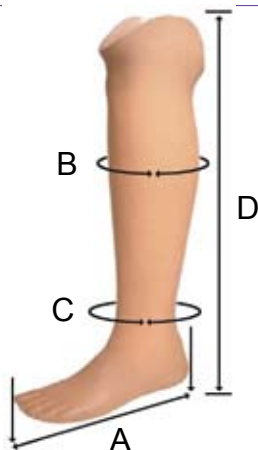

Features & Benefits

CENTRI AB has a long experience of cosmetic solutions for functional prosthesis. We can offer a BK skin with a natural appearance in the revolutionary material MCV. This material is very durable with good tear resistance. The unique surface of MCV gives low friction against fabrics and more resistance towards dirt and stains!

Size and Color

Dream Skin is available in 5 foot sizes, 21cm - 30 cm, and you can choose from 18 different skin colors. (Use Centri Color Guide)

Delivery

Dream Skin is delivered expanded for easy donning. After Dream Skin is in position you just simply apply heat to the material, max. 100° C. The skin will shrink to the under laying cosmesis.

Combinations

When you combine Dream Skin with the Quality Suspension Sleeve, you not only get excellent suspension of the prosthesis, but you also have an perfect solution for shower and bath!

Instructions	
To make donning easier, all Dream Skin are expanded and has talcum on the inside.	
1	Apply Dream Skin to the prosthesis. Stretch out the material to avoid wrinkles. Alt 1: Put the prosthesis inside an oven set on 100° C. Note! Make sure the prosthesis is standing away from any warm objects. Alt 2: Use a heat gun. Distribute the heat evenly over the surface of the Dream Skin!
2	Dream Skin will retract to its original shape and adjusts to the cosmesis.
3	Stretch the material and glue to the proximal brim. Glue Instructions: The sockets proximal part should be sandpapered to get a rough surface, carefully remove all sandpaper dust. Mix the adhesive thoroughly. Clean the parts of the Dream Skin that are going to be glued together carefully with alcohol. Apply the adhesive on both surfaces and let it dry for 1 minute to 30 minutes. Thereafter, press the surfaces together instantaneously with the highest possible pressure. No corrections can be done after the pressure has been applied, due to the adhesives quick binding time. The adhesive is tough enough for details to be handled right after the pressure has been applied. However the maximum strength of the adhesive is after 1-2 days.
4	Cut off surplus material proximal to the brim.

Accessories	
Product Nbr.	Description
611 200 002	Color Guide C01 - 18 with viewing hole.
631 150 001	Adhesive for Dream Skin and PVC/MCV Gloves 22ml
The adhesive also works on materials such as metal, leather, wood, textiles and rubber. The adhesive is flexible and resistant of softeners, oil, petrol and water.	



A Fillauer Company