

## PARYDONN LINER

The Freedom PARYDONN Liner™ is the industry's first medical-grade socket interface that is hygienic, odor free, durable, and easy to use. Manufactured using proprietary, medical grade platinum-cured silicone and Parylene Coating, the non-porous make-up of the liner prevents bacteria from entering and becoming trapped inside the material. As a result, odor is eliminated and healthy skin is promoted.

The Freedom PARYDONN Liner™ introduces Parylene Coating. Due to the coating's extremely low coefficient of friction, a thin layer of Parylene on the inside of the liner allows it to glide on and off the amputee's residual limb quickly, easily, and comfortably, without the use of a secondary lubricant. An outside coating efficiently replaces the need for fabric covers.

The Freedom PARYDONN Liner is available in eight sizes, both as a cushion liner and locking liner, and is suitable for use by most trans-tibial amputees.

- Manufactured Using Platinum-Cured
- Silicone
- Parylene Coating as a Donning and Doffing Agent
- Parylene Coating as a Sealing Agent
- Available both as a Cushion Liner and
- Locking Liner
- Tapered Wall Thickness Profile
- Thickness identical to ICEROSS
- Comfort Liner
- 8 sizes



The product is sold with a 6 month warranty. Return policy 60 days if unopened!

## Ordering Information

### To determine the correct liner size:

Take a circumferential measurement 4 cm from distal end of the stump. Then choose the liner which corresponds equally to the measurement OR the nearest smaller sizes from Paradonn Size Chart. Choose a Locking Liner or a Cushion Liner.

Circumferential Measurements 4 cm from Distal End (cm)	Ordering Information	
	Cushion Liner	Locking Liner
22	701 000 222	701 000 422
23,5	701 000 223	701 000 423
25	701 000 225	701 000 425
26,5	701 000 226	701 000 426
28	701 000 228	701 000 428
30	701 000 230	701 000 430
32	701 000 232	701 000 432
34	701 000 234	701 000 434

### Liner care

The amputee should be instructed to remove the liner, turn it inside out and wash with a mild soap. After cleaning, rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.